




Introducing IBM iConnect Access Physician Imaging Portal

Imaging Partners has invested in this upgrade for physicians to access reports and view images more efficiently. iConnect Access is a true zero download web viewer available in any browser.

1.

How to access iConnect Access

Login using username and password

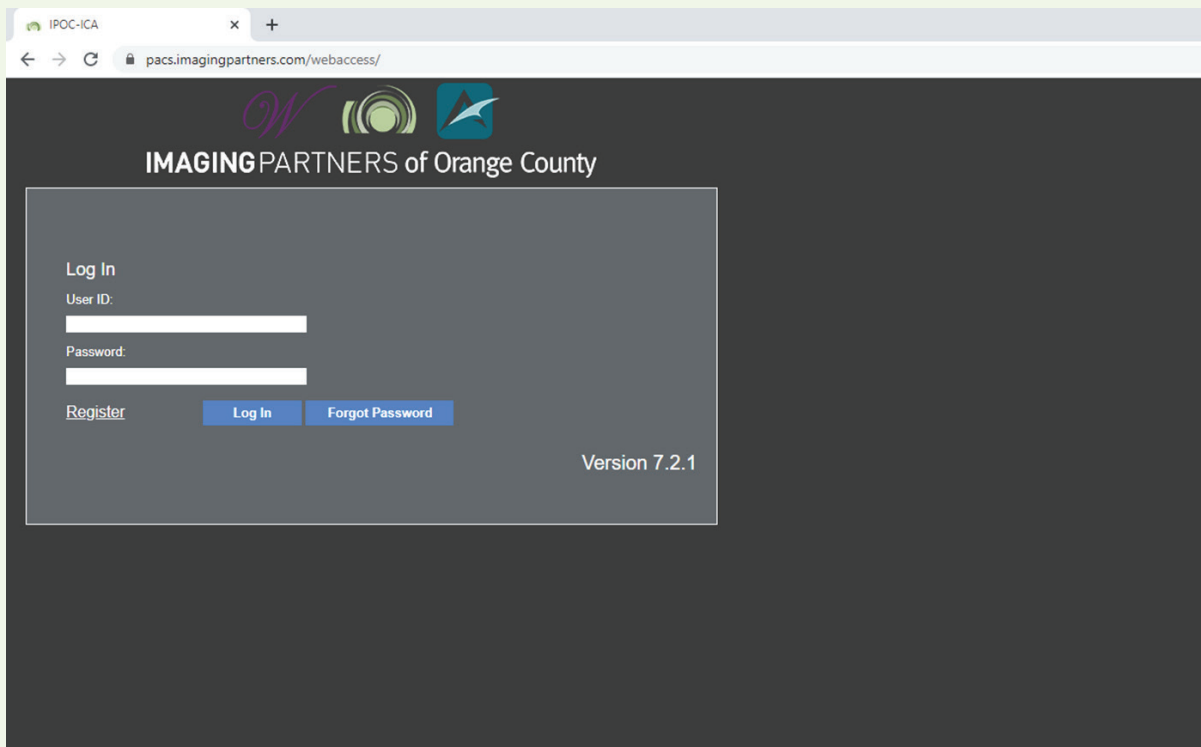

The screenshot shows a web browser window with the address bar displaying "pacs.imagingpartners.com/webaccess/". The page header features the "IMAGING PARTNERS of Orange County" logo. The main content area is a dark gray box containing a "Log In" section with "User ID:" and "Password:" labels, each followed by a white input field. Below the input fields are three links: "Register", "Log In", and "Forgot Password". The "Log In" and "Forgot Password" links are highlighted with blue backgrounds. The version number "Version 7.2.1" is displayed in the bottom right corner of the login box.




2.

Search for patient's images

Type your patient's first and last name.

Be sure the "study performed" field is set to the proper parameters. Then, press enter or click search.

Patient's first and last name

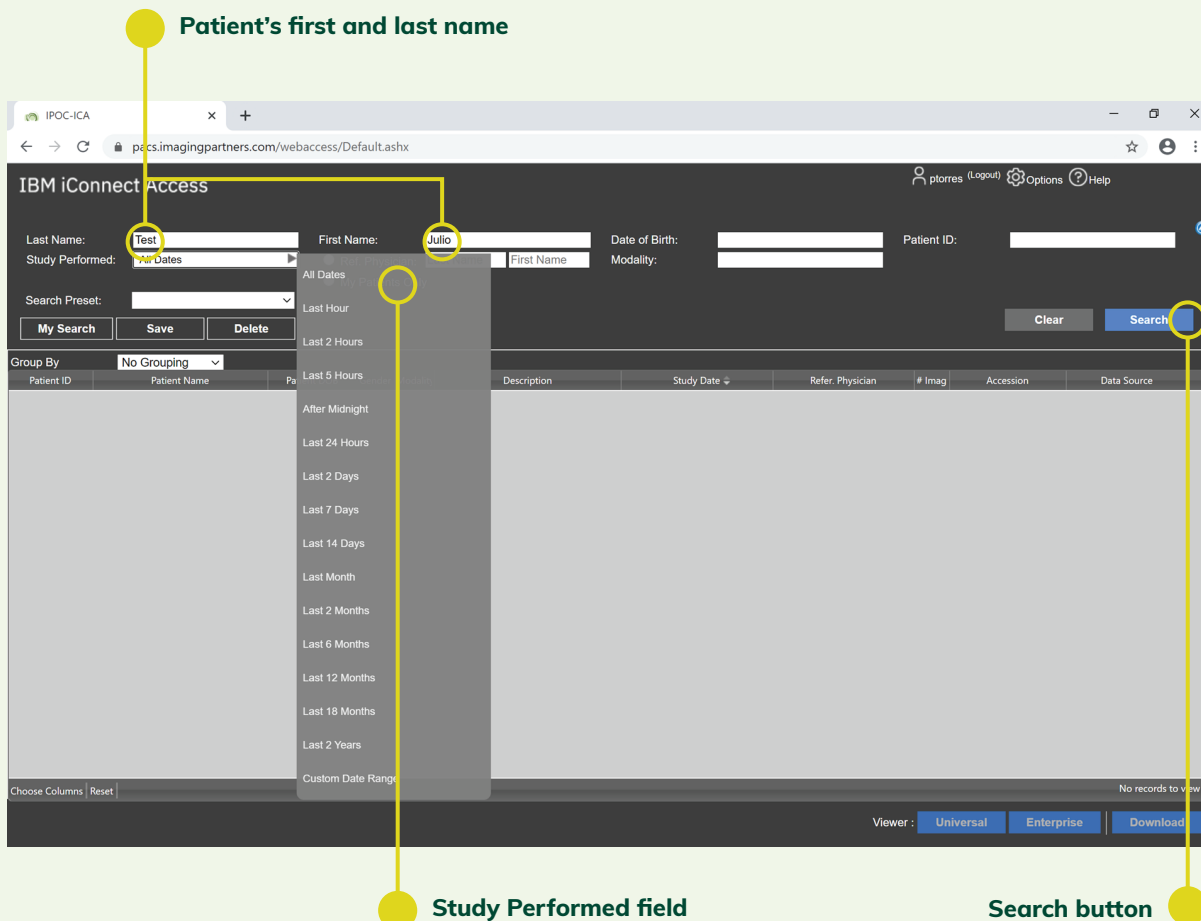

IBM iConnect Access

Last Name: First Name: Date of Birth: Patient ID:

Study Performed: First Name:

Search Preset:

Group By:

Description	Study Date	Refer. Physician	# Imag	Accession	Data Source
No records to view					

Viewer:

Study Performed field

Search button




3.

View patient's images

Double click name to view patient images

Patient name

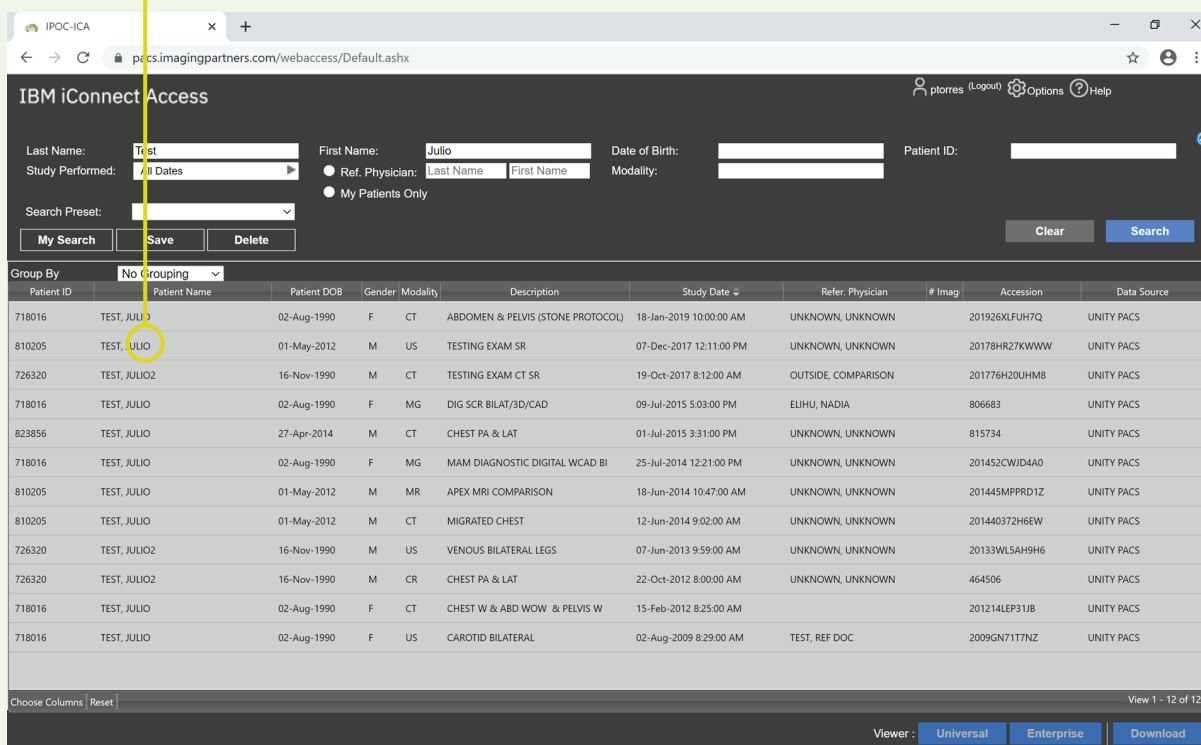

IBM iConnect Access

Last Name: First Name: Date of Birth: Patient ID:

Study Performed: Ref. Physician: Modality:

Search Preset:

Group By:

Patient ID	Patient Name	Patient DOB	Gender	Modality	Description	Study Date	Refer. Physician	# Imag	Accession	Data Source
718016	TEST, JULIO	02-Aug-1990	F	CT	ABDOMEN & PELVIS (STONE PROTOCOL)	18-Jan-2019 10:00:00 AM	UNKNOWN, UNKNOWN		201926XLFUH7Q	UNITY PACS
810205	TEST, JULIO	01-May-2012	M	US	TESTING EXAM SR	07-Dec-2017 12:11:00 PM	UNKNOWN, UNKNOWN		20178HR27KWWWW	UNITY PACS
726320	TEST, JULIO2	16-Nov-1990	M	CT	TESTING EXAM CT SR	19-Oct-2017 8:12:00 AM	OUTSIDE, COMPARISON		201776H20UHM8	UNITY PACS
718016	TEST, JULIO	02-Aug-1990	F	MG	DIG SCR BILAT/3D/CAD	09-Jul-2015 5:03:00 PM	ELIHU, NADIA		806683	UNITY PACS
823856	TEST, JULIO	27-Apr-2014	M	CT	CHEST PA & LAT	01-Jul-2015 3:31:00 PM	UNKNOWN, UNKNOWN		815734	UNITY PACS
718016	TEST, JULIO	02-Aug-1990	F	MG	MAM DIAGNOSTIC DIGITAL WCAD BI	25-Jul-2014 12:21:00 PM	UNKNOWN, UNKNOWN		201452CWID4A0	UNITY PACS
810205	TEST, JULIO	01-May-2012	M	MR	APEX MRI COMPARISON	18-Jun-2014 10:47:00 AM	UNKNOWN, UNKNOWN		201445MPPRD1Z	UNITY PACS
810205	TEST, JULIO	01-May-2012	M	CT	MIGRATED CHEST	12-Jun-2014 9:02:00 AM	UNKNOWN, UNKNOWN		201440372H6EW	UNITY PACS
726320	TEST, JULIO2	16-Nov-1990	M	US	VENOUS BILATERAL LEGS	07-Jun-2013 9:59:00 AM	UNKNOWN, UNKNOWN		20133WLSAH9H6	UNITY PACS
726320	TEST, JULIO2	16-Nov-1990	M	CR	CHEST PA & LAT	22-Oct-2012 8:00:00 AM	UNKNOWN, UNKNOWN		464506	UNITY PACS
718016	TEST, JULIO	02-Aug-1990	F	CT	CHEST W & ABD WCW & PELVIS W	15-Feb-2012 8:25:00 AM			201214LEP31JB	UNITY PACS
718016	TEST, JULIO	02-Aug-1990	F	US	CAROTID BILATERAL	02-Aug-2009 8:29:00 AM	TEST, REF DOC		2009GN717TNZ	UNITY PACS

Choose Columns | Reset | View 1 - 12 of 12

Viewer:




4.

How to view series of images

To scroll through the series of images you can use your mouse wheel or left click and drag the image number slide bar.

Image number slide bar

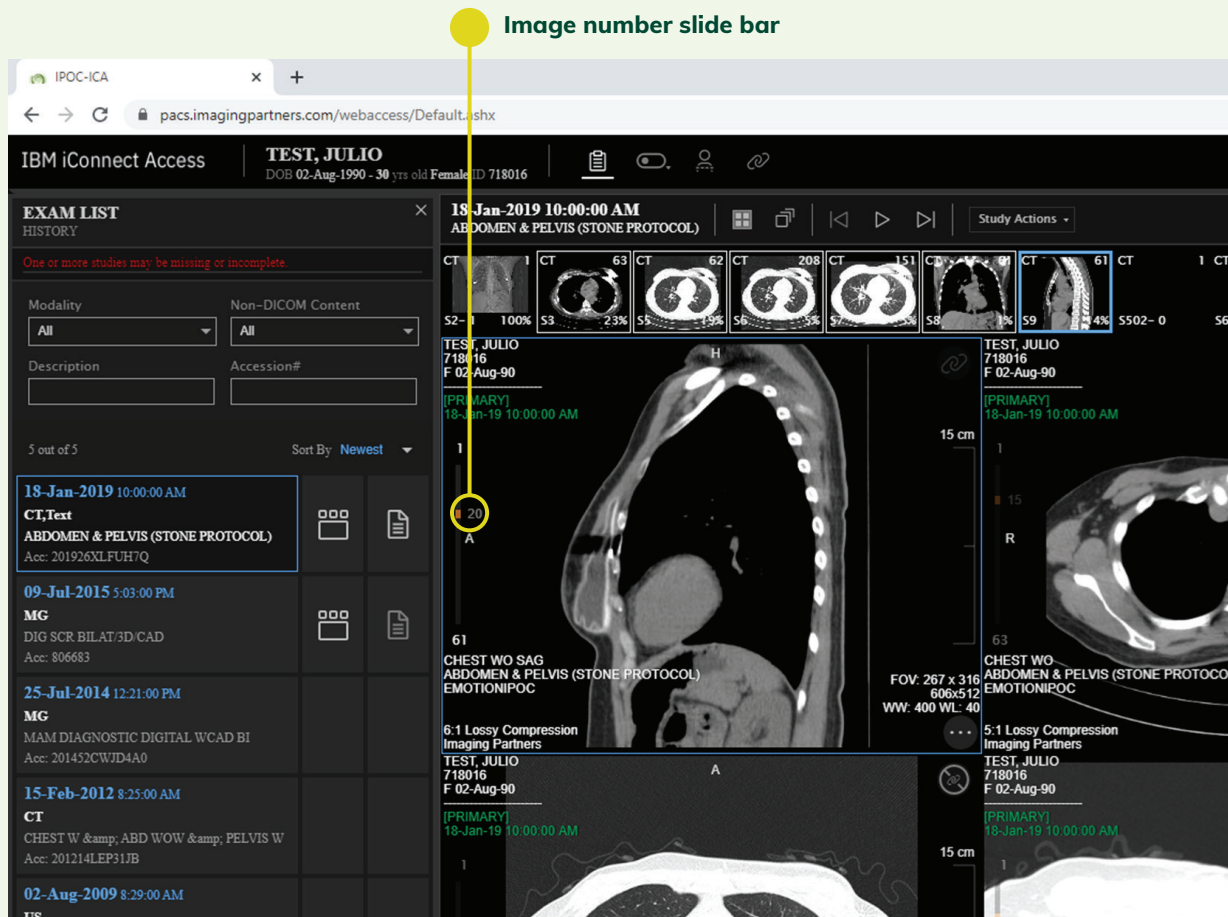


The screenshot displays the IBM iConnect Access PACS interface for patient TEST, JULIO (DOB 02-Aug-1990, 30 YRS old Female, ID 718016). The left pane shows the EXAM LIST HISTORY with 5 out of 5 exams. The selected exam is from 18-Jan-2019 10:00:00 AM, CT, ABDOMEN & PELVIS (STONE PROTOCOL), Accession# 201926XLFUH7Q. The right pane shows a series of CT images. A yellow circle highlights the image number slide bar, which is a vertical bar with a slider, used for navigating through the series of images. The main image area shows a sagittal view of the abdomen and pelvis.



5.

How to use image tools

To access different tools, simply right click anywhere on the image and select your tool preference.

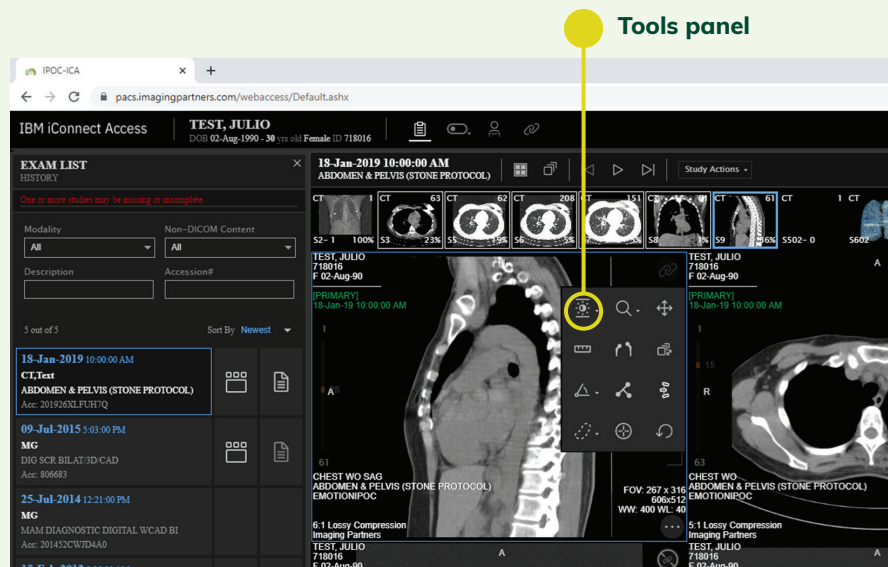

Tools panel

6.

How to change viewer layout

To alter the image viewer layout, click the layout button and select your preference.

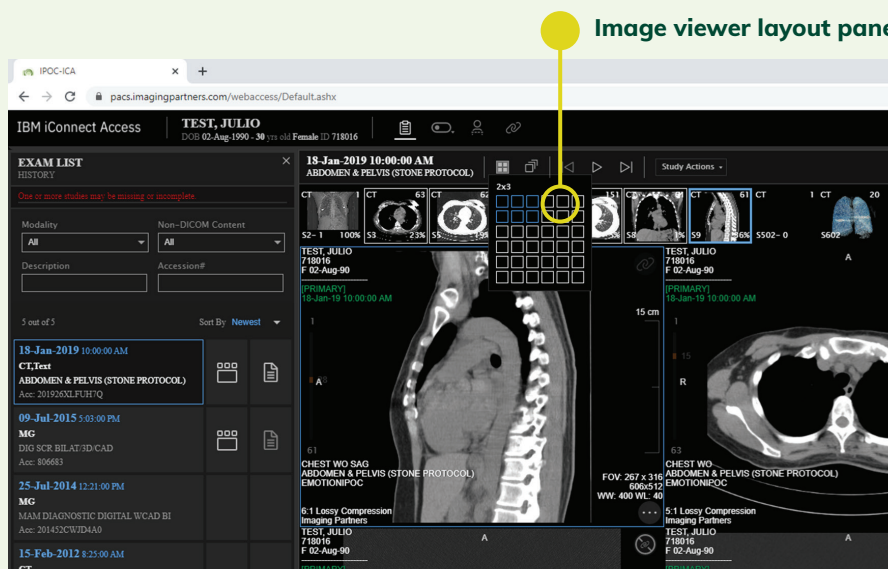



Image viewer layout panel



7.

View multiple series of images


To view a different series of images, left click and drag the image series thumbnail to the desired view box.

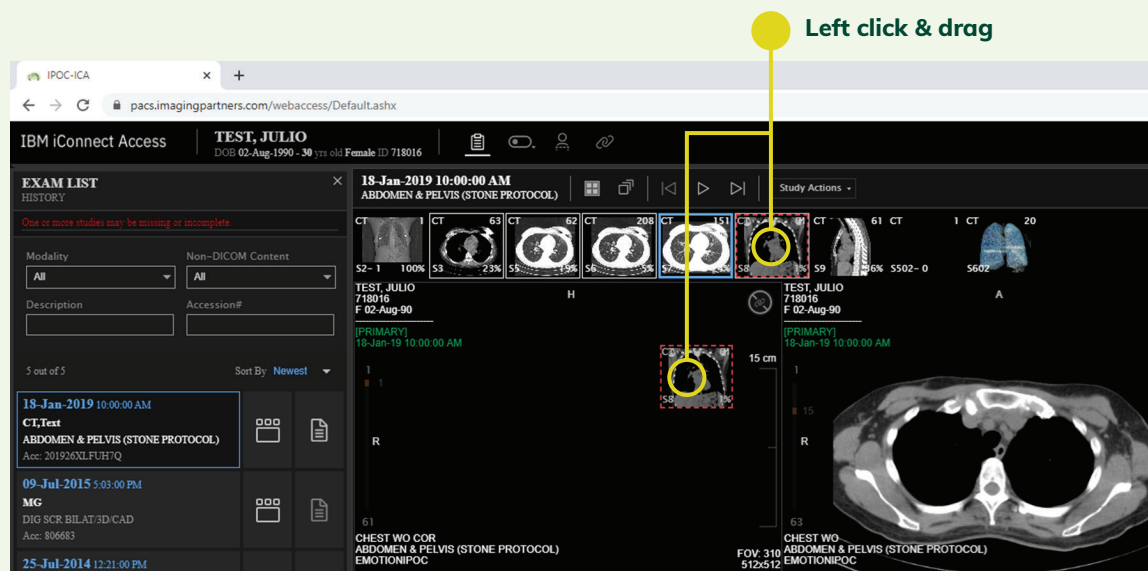

8.

Turn on/off localizer

You can toggle the appearance of localizer lines on and off by clicking the localizer lines button.

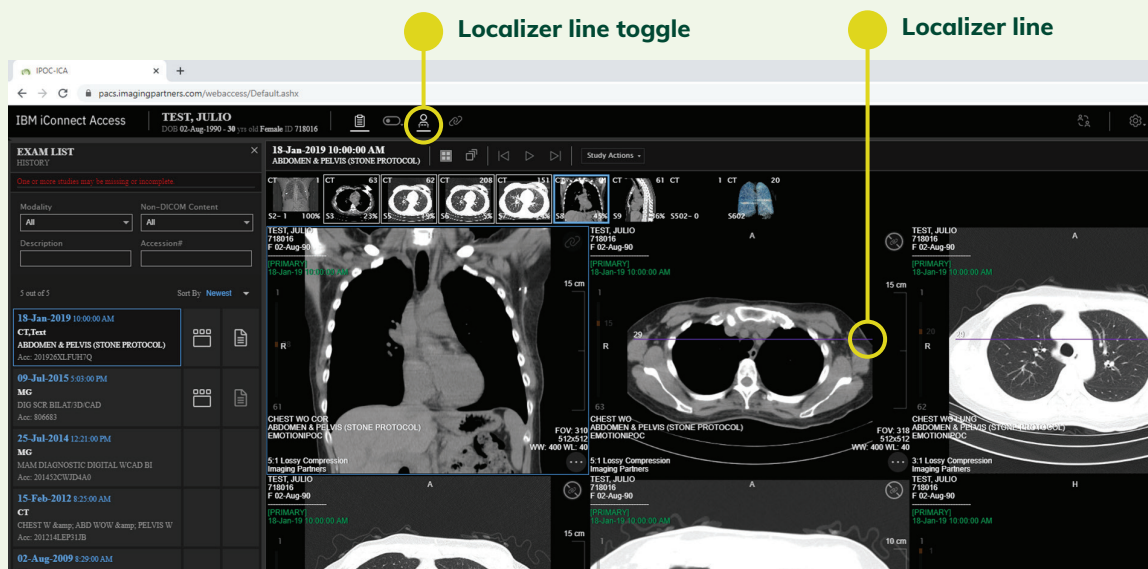




9.

How to link image


Some image series can be linked. To link an image series, click the link scroll button and continue to scroll through an image series.

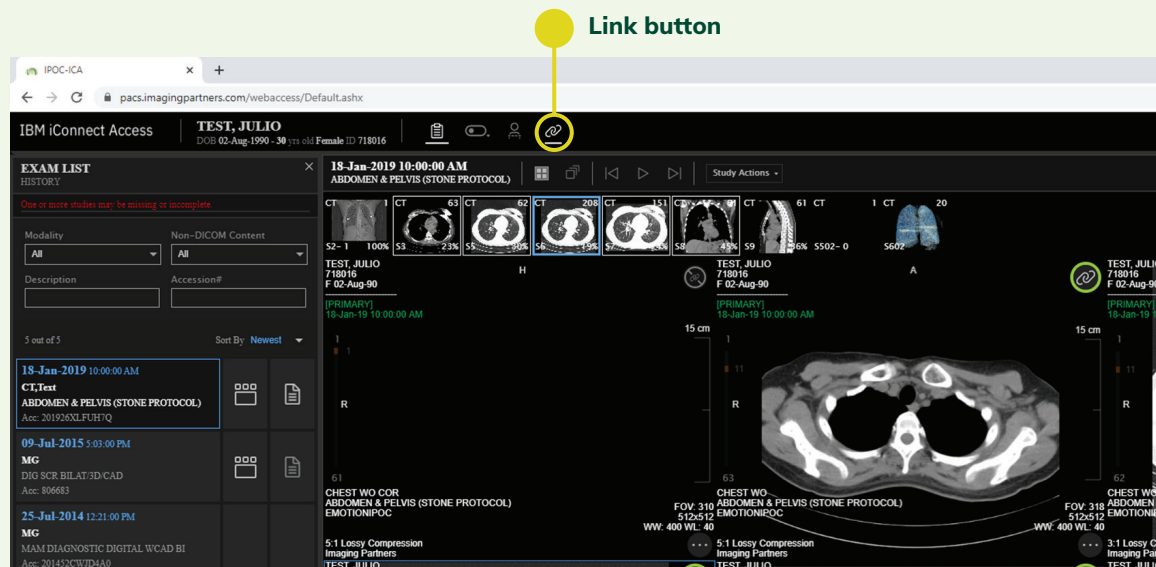

10.

Linked images

The green circle will indicate which images are linked.

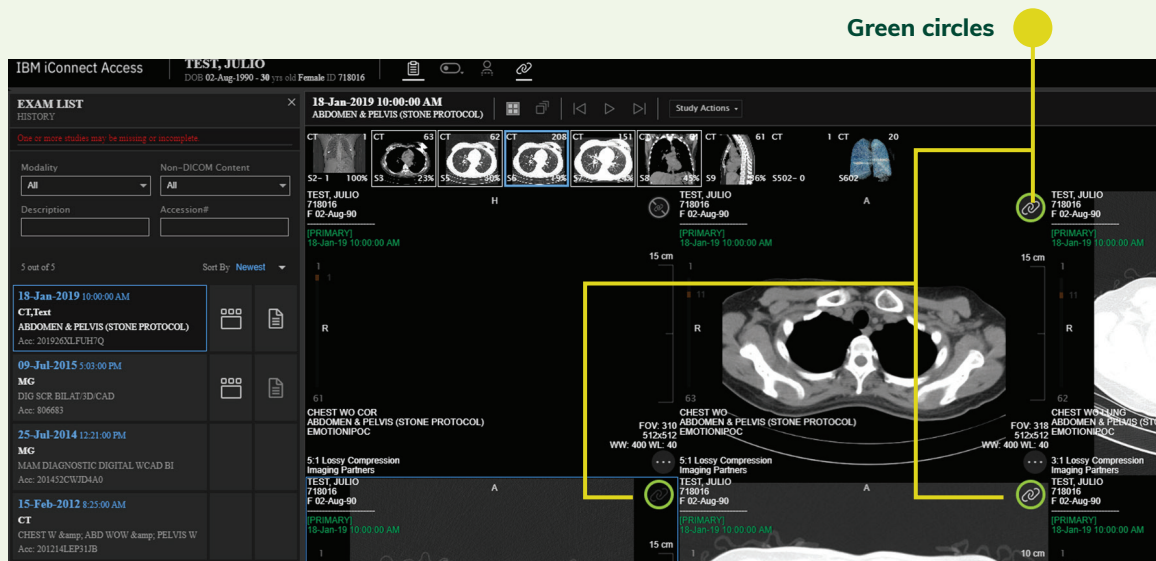




11. View exam report

Click the exam report button to review the radiologist's findings.

Exam report button

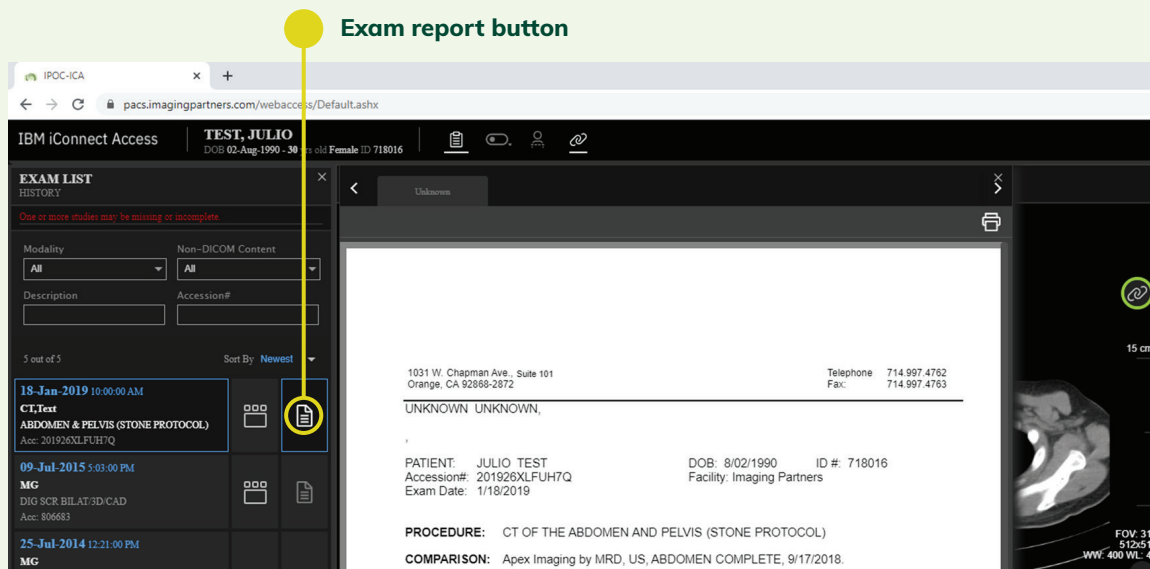


The screenshot shows the IBM iConnect Access interface. On the left, the 'EXAM LIST' table displays a list of exams. The first exam, dated 18-Jan-2019 10:00:00 AM, is highlighted. A yellow circle highlights the 'Exam report button' (a document icon) in the 'Actions' column of this row. The main panel displays patient information for 'TEST, JULIO' (DOB: 02-Aug-1990 - 30 yrs old Female ID 718016) and the exam details: 'CT, Text ABDOMEN & PELVIS (STONE PROTOCOL)' (Acc: 201926XLFUH7Q). The procedure is 'CT OF THE ABDOMEN AND PELVIS (STONE PROTOCOL)' and the comparison is 'Apex Imaging by MRD, US, ABDOMEN COMPLETE, 9/17/2018'.

12. How to print report

To print the report, click the print button.

Print button

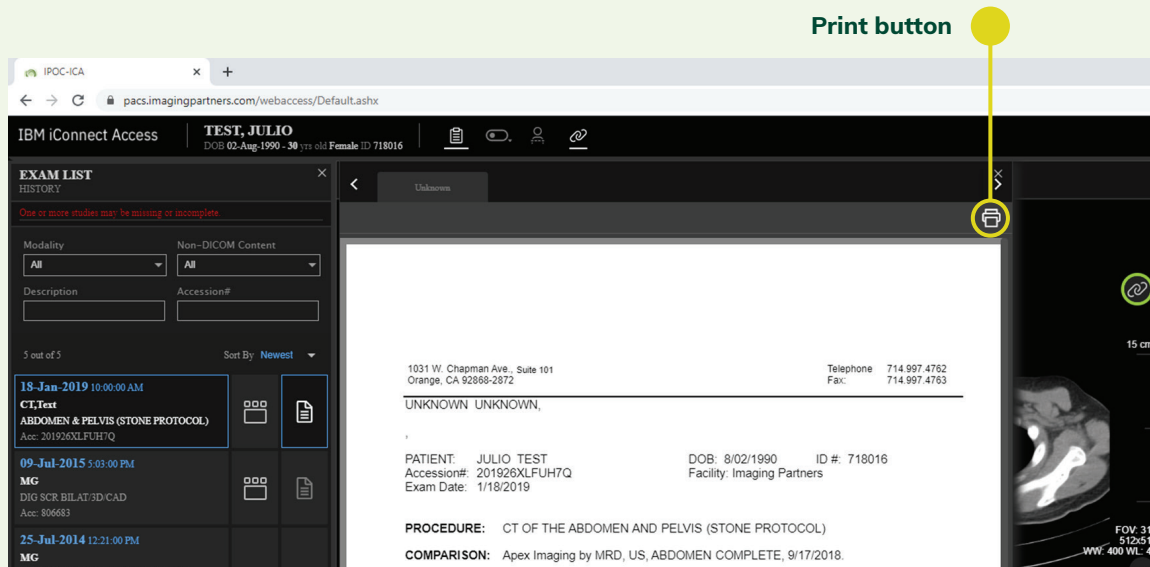

The screenshot shows the IBM iConnect Access interface. On the left, the 'EXAM LIST' table displays a list of exams. The first exam, dated 18-Jan-2019 10:00:00 AM, is highlighted. A yellow circle highlights the 'Print button' (a printer icon) in the 'Actions' column of this row. The main panel displays patient information for 'TEST, JULIO' (DOB: 02-Aug-1990 - 30 yrs old Female ID 718016) and the exam details: 'CT, Text ABDOMEN & PELVIS (STONE PROTOCOL)' (Acc: 201926XLFUH7Q). The procedure is 'CT OF THE ABDOMEN AND PELVIS (STONE PROTOCOL)' and the comparison is 'Apex Imaging by MRD, US, ABDOMEN COMPLETE, 9/17/2018'.



13.

Exit exam and log out

To ensure the privacy of patients, please be sure to exit the exam and log out when finish viewing.

